2. Abzug, 29. Januar 2006

StorAX Cary-Blair

Probenröhrchen für Stuhlkulturen
Tubes d'échantillons pour la culture des selles
Provette per coprocultura
Sample tubes for fecal cultures

Gebrauchsanweisung Notice d'utilisation Istruzioni per l'uso Instruction for use

**REF**

AL0302

Cont.

- 25 Stuhlröhrchen mit Cary-Blair-Lösung/
Tubes d'échantillon de selles avec
solution Cary-Blair/Provette per la
raccolta di feci con soluzione Cary-
Blair/Stool collection tubes with a
Cary-Blair solution
- 25 Patientenetiketten/Etiquettes pour
patient/Etichette paziente/Patient labels
- 1 Gebrauchsanweisung/Notice
d'utilisation/Istruzioni per l'uso/
Instruction for use

In vitro diagnosticum
Usage in vitro
Per uso diagnostico in vitro
For in vitro diagnostic use

Axon Lab AG, CH-5405 Baden

StorAX Cary-Blair

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Bibliografia

1. Cary and Blair: J. Bacteriol. 88:96, 1964
2. NCCLS Publication: Quality Assurance Standards for Commercially Prepared Microbiological Culture Media Volume 10:14,1990

StorAX Cary-Blair

Intended use

Transport medium for preserving pathogenic intestinal bacteria.

Package contents

- 25 tubes with spoon, brown screw cap and 7.5 ml of Cary-Blair solution.
- Instructions for use

Principle

StorAX Cary-Blair is designed to preserve pathogenic intestinal bacteria while the sample is transported.

Composition


Basic composition in %

Disodium phosphate	0.1
Sodium chloride	0.5
Sodium thioglycolate	0.15
pH	8.5

Storage

The StorAX Cary-Blair tubes are stored at 15–30 °C. Do not freeze. Do not use the product after the expiration date specified on the package.

Obtaining stool samples

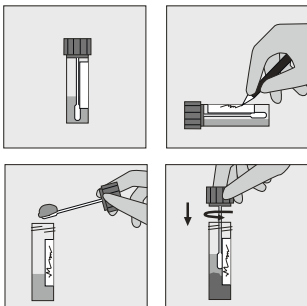Use the spoon to place the freshly obtained stool sample into the Cary-Blair transport medium. A spoon-sized sample is sufficient.

Transporting stool samples with StorAX Cary-Blair

The stool samples can be transported at 15–30 °C. Laboratory processing should occur within 72 hours.

Quality control

Quality controls are carried out during the manufacture of every Cary-Blair lot.

Disposal

All laboratories are responsible for disposal of waste in accordance with the legal requirements.

References

1. Cary and Blair: J. Bacteriol. 88:96, 1964
2. NCCLS Publication: Quality Assurance Standards for Commercially Prepared Microbiological Culture Media Volume 10:14,1990

Abkürzungen/Abréviations/Abbreviazioni/Abbreviations Symbole/Symbole/Simboli/Symbols

Symbol/Symbole/Simbolo	Bedeutung/Signification/ Significato
	Bestellnummer/Numéro d'article/ Numero di catalogo/Catalogue number
	Packungsinhalt/Contenu du coffret/ Contenuto della confezione/Contents of kit
	In vitro diagnosticum/Usage in vitro/Per uso diagnostico in vitro/For in vitro diagnostic use
	Hersteller/Fabricant/Fabbricante/Manufacturer
	Lagertemperatur/Température de conservation/ Temperatura di conservazione/Storage temperature
	Verwendbar bis/Date de péremption/ Data di scadenza/Use before
	Chargenbezeichnung / Numéro du lot/ Codice del lotto/Lot number
	Gebrauchsanweisung beachten/Consulter le mode d'emploi/Leggere le istruzioni per l'uso/Consult the package insert
	Einmalgebrauch/Usage unique/ Per uso unico/Single use.
	CE-Kennzeichnung/CE marquage/ Marchio CE/CE marking

Schweiz

Axon Lab AG
Täfernstrasse 15
CH-5405 Baden
Tel. +41 (0)56 484 80 80
Fax +41 (0)56 484 80 99
info@axonlab.ch
www.axonlab.ch

Suisse

Axon Lab SA
En Budron E9
CH-1052 Le Mont-sur-Lausanne
Tel. +41 (0)21 651 71 50
Fax +41 (0)21 651 71 59
info_f@axonlab.ch

Deutschland

Axon Lab AG
Heinrich-Otto-Strasse 1
D-73262 Reichenbach/Stuttgart
Tel. +49 (0)7153 9226-0
Fax +49 (0)7153 9226-26
info@axonlab.de
www.axonlab.de

Österreich

Axon Lab AG
Hafenstrasse 47-51
A-4020 Linz
Tel. +43 (0)732 901 561 00
Fax +43 (0)732 901 561 19
info@axonlab.at
www.axonlab.at